D

Gebrauchsinformation

LAVANOX-Serag® Wundgel

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SERAG-WIESSNER GmbH & Co. KG
Zum Kugelfang 8-12, D-95119 Naila

Hersteller:

Prontomed GmbH
Am Bahndamm 70
D-32120 Hiddnhausen
Made in Germany

Instructions for use

LAVANOX-Serag® wound gel

Intended use:

LAVANOX-Serag® wound Gel is a ready-to-use hydrogel for low-pain application to acute and chronic wounds. It forms a protective film, keeps the wound moist and thus promotes wound healing. LAVANOX-Serag® wound Gel is used to cleanse moisten and keep wounds moist. It forms an ideal wound healing environment and thus supports the natural healing process in the wound. Wound odors are quickly and reliably eliminated using LAVANOX-Serag® wound gel.

Product properties:

The application of LAVANOX-Serag® wound gel is non-irritating, and virtually painless. There is no restriction on the frequency and duration of use. Residues of LAVANOX-Serag® Wound Gel may remain in the wound. As a multi-dose container, LAVANOX-Serag®

wound gel is preserved with NaOCl/HOCl. This allows the use of the opened individual product for 12 weeks.

Application information:

Shake tube well before use! Before the first application and at dressing changes, it is recommended to use a wound spray or wound irrigation solution to remove exudate, fibrinoid deposits and other contaminants. Suitable for this purpose are products of the line LAVANOX-Serag®. The frequency of dressing changes depends on the nature of the wound and should be adapted to the particular wound situation. Generously cover the wound surface with LAVANOX-Serag® Wound Gel and dress the wound for protection. LAVANOX-Serag® wound gel can be used in combination with common compresses, plasters and other common dressing materials. Please also refer to the instructions for use of the respective product. LAVANOX-Serag® wound gel can be used without restriction regarding age and gender of the patient. There are no known undesirable side effects. A slight chlorine odor is a sign of the activity of the preservative system. LAVANOX-Serag® wound gel can be used by non-professionals if they have been instructed by healthcare professionals.

Application restrictions:

LAVANOX-Serag® wound gel should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Do not use simultaneously with products containing chlorhexidine. Do not use LAVANOX-Serag® wound gel on bleeding or exuding wounds. For use during pregnancy, while breast-feeding and in children under two years of age, consult your doctor or pharmacist beforehand. In children up to 12 years of age, the application

should be supervised by an adult. If there is redness, swelling or pain in the wound area or if the condition of the wound does not improve within a few days, a doctor should be consulted.

Safety instructions:

Do not swallow. Avoid contact with eyes. Keep safe from children. Use only tubes that are in perfect condition and undamaged. Use the product only until the stated expiry date and not longer than 12 weeks after opening. Please report a potentially serious incident involving the product immediately to Promotmed (vigilanz@promotmed.de) and the competent authority in your country.

Ingredients:

Aqua, <0,06% sodium hypochlorite/hypochlorous acid, Lithium magnesium sodium silicate

Issue: V01/2022.08

Distributed by:

SERAG-WIESSNER GmbH & Co. KG
Zum Kugelfang 8-12, D-95119 Naila

Manufacturer:

Promotmed GmbH
Am Bahndamm 70
D-32120 Hiddenhausen
Made in Germany

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