D

Gebrauchsinformation

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SERAG-WIESSNER GmbH & Co. KG
Zum Kugelfang 8-12, D-95119 Naila

Hersteller:

Prontomed GmbH
Am Bahndamm 70
D-32120 Hiddnhausen
Made in Germany

Instructions for use

LAVANOX-Serag® wound spray gel

Intended use:

LAVANOX-Serag® wound spray gel is used for cleansing, long-term moisturizing and keeping acute and chronic wounds moist. It forms a protective film, keeps the wound moist and thus promotes wound healing. LAVANOX-Serag® wound spray gel forms an ideal wound healing environment, supporting the natural wound healing process. Wound odors are quickly and reliably eliminated using LAVANOX-Serag® wound spray gel.

Product properties:

Due to its properties, LAVANOX-Serag® wound spray gel can be applied well and quickly, without any additional aids, and remains as a hydrogel in the

wound. The application of LAVANOX-Serag® wound spray gel is non-irritating, and virtually painless. There is no restriction on the frequency and duration of use. As a multi-dose container, LAVANOX-Serag® wound spray gel is preserved with NaOCl/HOCl. This allows the use of the opened individual product for 12 weeks.

Application information:

Shake bottle well before use!
Spray LAVANOX-Serag® wound spray gel with several bursts from a short distance until the wound is completely wetted, depending on its nature. Before first application and at dressing changes, wound cleansing is recommended to remove exudate, fibrinoid deposits and other contaminants. Suitable for these are products of the line LAVANOX-Serag®. The frequency of dressing changes depends on the nature of the wound and should be adapted to the particular wound situation. LAVANOX-Serag® wound spray gel can be used in combination with common compresses, plasters and other common dressing materials. Please also refer to the instructions for use of the respective product. LAVANOX-Serag® wound spray gel can be used without restriction regarding age and gender of the patient. There are no known undesirable side effects. A slight chlorine odor is a sign of the activity of the preservative system. LAVANOX-Serag® wound spray gel can be used by non-professionals if they have been instructed by healthcare professionals.

Application restrictions:

LAVANOX-Serag® wound spray gel should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Do not use simultaneously with products containing chlorhexidine. Do not use LAVANOX-Serag®

wound spray gel on bleeding or exuding wounds. For use during pregnancy, while breast-feeding and in children under two years of age, consult your doctor or pharmacist beforehand. In children up to 12 years of age, the application should be supervised by an adult. If there is redness, swelling or pain in the wound area or if the condition of the wound does not improve within a few days, a doctor should be consulted.

Safety instructions:

Do not swallow. Avoid contact with eyes. Keep safe from children. Use only bottles that are in perfect condition and undamaged. Use the product only until the stated expiry date and not longer than 12 weeks after opening. Please report a potentially serious incident involving the product immediately to [**Prontomed \(vigilanz@prontomed.de\)**](mailto:Prontomed (vigilanz@prontomed.de)) and the competent authority in your country.

Ingredients:

Aqua, <0,06% sodium hypochlorite/hypochlorous acid, lithium magnesium sodium silicate

Issue: V01/2022.08

Distributed by:

SERAG-WIESSNER GmbH & Co. KG
Zum Kugelfang 8-12, D-95119 Naila

Manufacturer:

Prontomed GmbH
Am Bahndamm 70
D-32120 Hiddenhausen
Made in Germany

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