**D**

## Gebrauchsinformation

# LAVANOX-Serag® Wundspüllösung 1000ml

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SERAG-WIESSNER GmbH & Co. KG  
Zum Kugelfang 8-12, D-95119 Naila

### **Hersteller:**

Promoted GmbH  
Am Bahndamm 70  
D-32120 Hiddnhausen  
Made in Germany

## Instructions for use

# LAVANOX-Serag® wound irrigation solution 1000ml

### **Intended use:**

LAVANOX-Serag® wound is used for cleansing, moisturising, mechanical decontamination and infection prophylaxis in acute and chronic wounds. During wound cleansing, LAVANOX-Serag® Wound irrigation solution helps to remove biofilm and fibrinoid debris. In addition, LAVANOX-Serag® wound irrigation solution supports a low-pain removal of wound dressings. Wound odors are quickly and reliably eliminated using LAVANOX-Serag® wound irrigation solution. Suitable for use for negative pressure wound therapy (NPWT).

### **Product properties:**

The application of LAVANOX-Serag® wound irrigation solution is non-irritating, and virtually painless. Residues of LAVANOX-Serag® wound irrigation solution may remain in the wound. There is no restriction on the frequency and duration of use. As a multi-dose container, LAVANOX-Serag® wound

irrigation solution is preserved with NaOCl/HOCl. This allows the use of the opened individual product for 12 weeks.

### **Application information:**

A wound should first be cleaned of dirt, blood and coatings. For this purpose, the affected areas are rinsed intensively with a generous amount of LAVANOX-Serag® wound irrigation solution. In case of heavy contamination, the application can be repeated several times. After the application of LAVANOX-Serag® wound irrigation solution, it is not necessary to rinse with another solution (i.e. residues of LAVANOX-Serag® wound irrigation solution may remain in the wound). Encrusted and difficult to remove wound dressings or bandages are richly soaked generously with LAVANOX-Serag® wound irrigation solution until they are soaked. After a short waiting time, they can be carefully removed with little pain. If necessary, a thin layer of wound gel can then be applied. The wound gel and the wound irrigation solution from the LAVANOX-Serag® product line are suitable for this. LAVANOX-Serag® wound irrigation solution can be used without restriction regarding age and gender of the patient. There are no known undesirable side effects. A slight chlorine odor is a sign of the activity of the preservative system. LAVANOX-Serag® wound irrigation solution can be used by non-professionals if they have been instructed by healthcare professionals. NPWT in combination with LAVANOX-Serag® wound irrigation solution may only be performed by healthcare professionals. Please refer to the instructions for use provided for your NPWT system.

### **Application restrictions:**

LAVANOX-Serag® wound irrigation solution should not be used if there are known allergies to any of the ingredients. For external use on skin wounds. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Do not use simultaneously with products containing chlorhexidine. Do not use LAVANOX-Serag® wound irrigation

solution on bleeding or exuding wounds. For use during pregnancy, while breast-feeding and in children under two years of age, consult your doctor or pharmacist beforehand. In children up to 12 years of age, the application should be supervised by an adult. If there is redness, swelling or pain in the wound area or if the condition of the wound does not improve within a few days, a doctor should be consulted.

#### Safety instructions:

Do not use for infusion or injection! Do not swallow. Avoid contact with eyes. Keep safe from children. Use only bottles that are in perfect condition and undamaged. Use the product only until the stated expiry date and not longer than 12 weeks after opening. Please report a potentially serious incident involving the product immediately to Prontomed (vigilanz@prontomed.de) and the competent authority in your country.

#### Ingredients:

Aqua, <0,08% sodium hypochlorite/ hypochlorous acid

Issue: V01/2022.08

#### Distributed by:

SERAG-WIESSNER GmbH & Co. KG  
Zum Kugelfang 8-12, D-95119 Naila

#### Manufacturer:

Prontomed GmbH  
Am Bahndamm 70  
D-32120 Hiddenhäusen  
Made in Germany

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